


Operation Manual

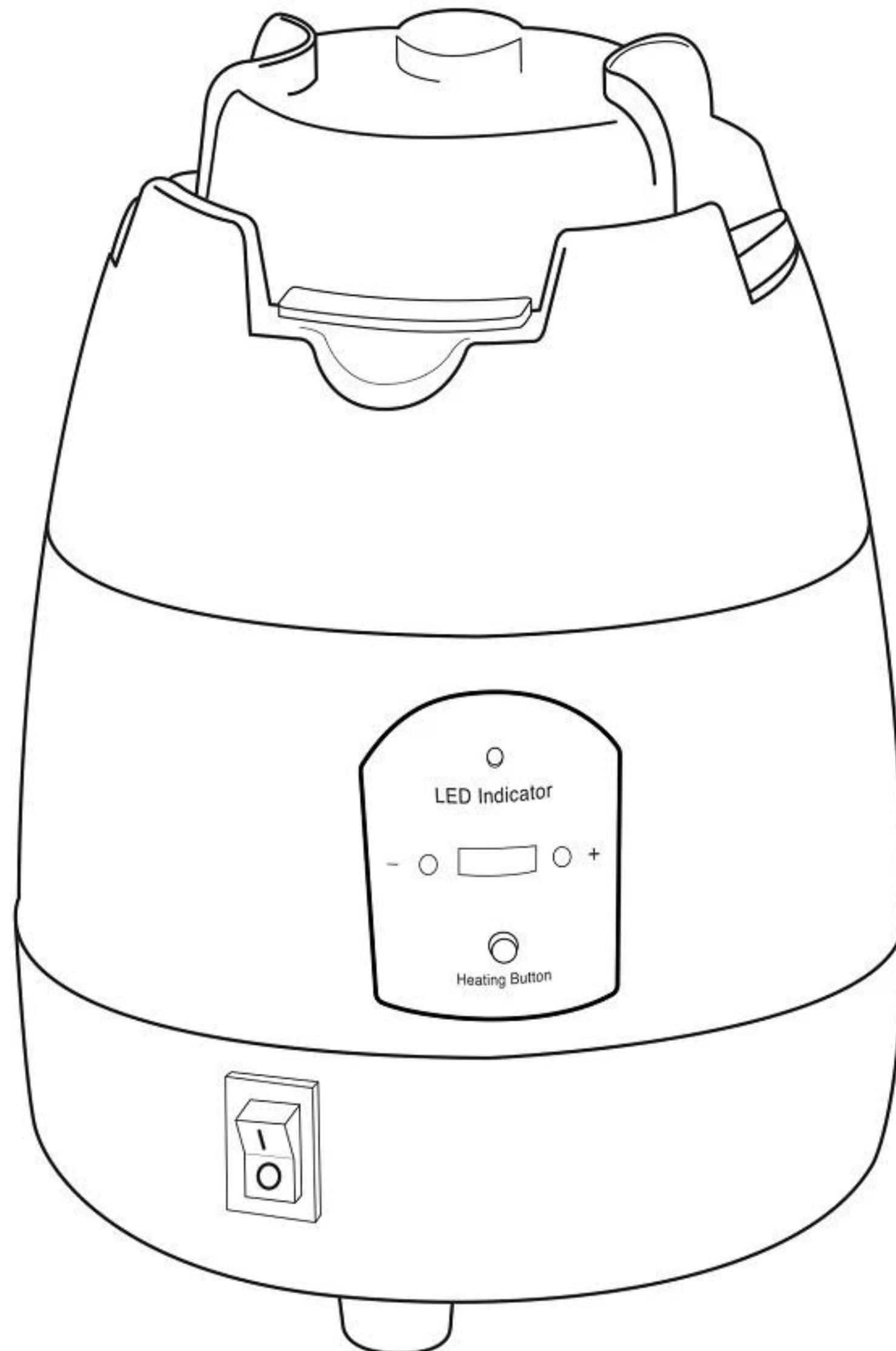

Dear Users:

Thank you very much for purchasing our heater. We strongly believe that quality and safety precedes all. In order to extend machine life, please read the items below with patience.

1. Switch on, press option button, heater works normally towards primitively setted 125° which is shown on the screen.

2. Press "+" or "-" button to increase or decrease the temperature by 5 degree each time, highest 150 , lowest 90° ,pls select suitable temperature.

3. After setted, Screen dispalys the change until to the stable setted temperature. Screen indicates "... " and heater turns to cold air if not being operated within 5 mins.

4. Press option button to shift to cold air, press again to primitively setted 125°


5. In the condition of cold air, the machine automatically stops within 2 mins, recommend to plug off.

CAUTIONS:

1. To extend use-life, do not plug off directly during heating.

2. When working abnormally, itself stops and stays dormant. Please return to factory, do not repair by yourself.

3. To prevent from short-circuit, do not use under moisturised circumstance.

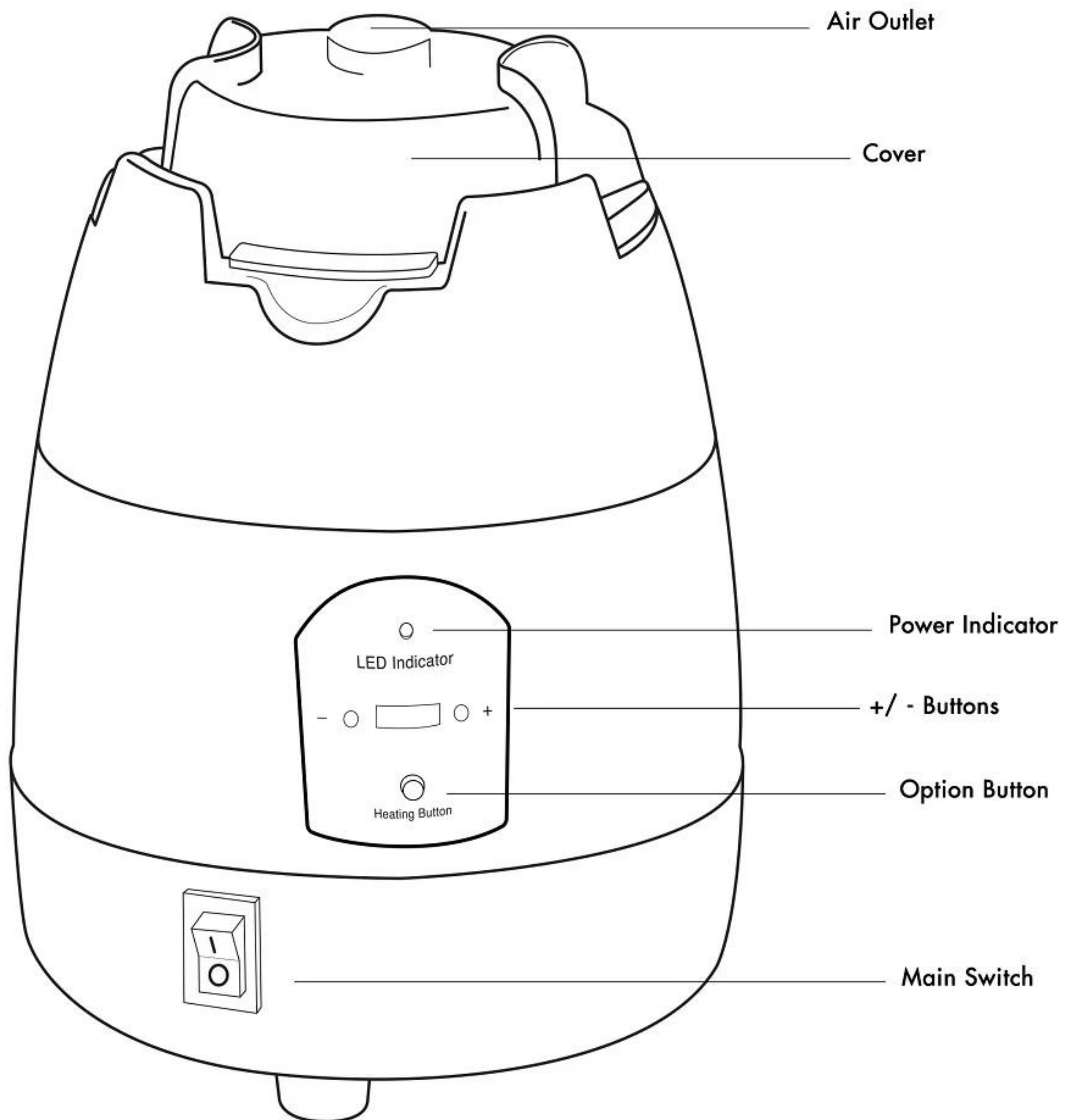

FH-6500 Frame Heater



9990 NW 14 Street | Suite 105 | Doral, FL 33172
Telephone: 888.334.4640