

# User's Manual



Dear Customers:

Congratulations!

You have purchased hand edger, it is really a good and correct choice of yours. In order to help you grasp the directions for using this machine and prolong its life, you are advised to read this manual with great care.

We reserve the right to change the information and instruction of the manual, we won't notice separately and do not bear the responsibility.



# Direction for Operation

When this machine is to be used, connect the hose with the down water tank, then place the down water tank into the bottom of the machine and then lead the drain pipe to the sewer.

Place the upper water tank on the top of the machine and put into some clean water. When the machine is used for the first time, take the sponge out from under the water valve, absorb enough water and then put it back to original place. Open the water valve and set the drop at 80-100 per minute, Then turn on the switch, the machine is ready to be used.

If model W wheel is chosen, the right side will be rough grinding, the left is fine grinding. When model WV wheel is chosen, the same with model W, just there is an extra V-shape to refine the lens.

After used for a period of time, take the down water tank out and clean it in case the draining hole would be blocked.

When the grinding wheel has been used for a period of time, grinding fatigue may appear. In this case, please turn on the water valve while use the oilstone stick to recondition the wheel.

## [Marks]

- If you buy the machine with reversible switch, when push the switch to I gear, the wheel will rotate in counter-clockwise; When push to the central, the machine will not work; When push to II gear, the wheel will rotate in clockwise.

- If you buy the machine with touching switch, when you edge the lens, just need touch the stainless steel piece, the machine will work automatically, after finishing and moving out, the machine will stop automatically.



# Unwrap the machine


1. Open the carton and take the machine wrapped in the foamed plastics.
2. Take the machine out carefully and place it on a big and steady working table.

## Accessories

Please give a check to the following accessories as soon as you take the machine out of the carton.

A hose

an oilstone stick

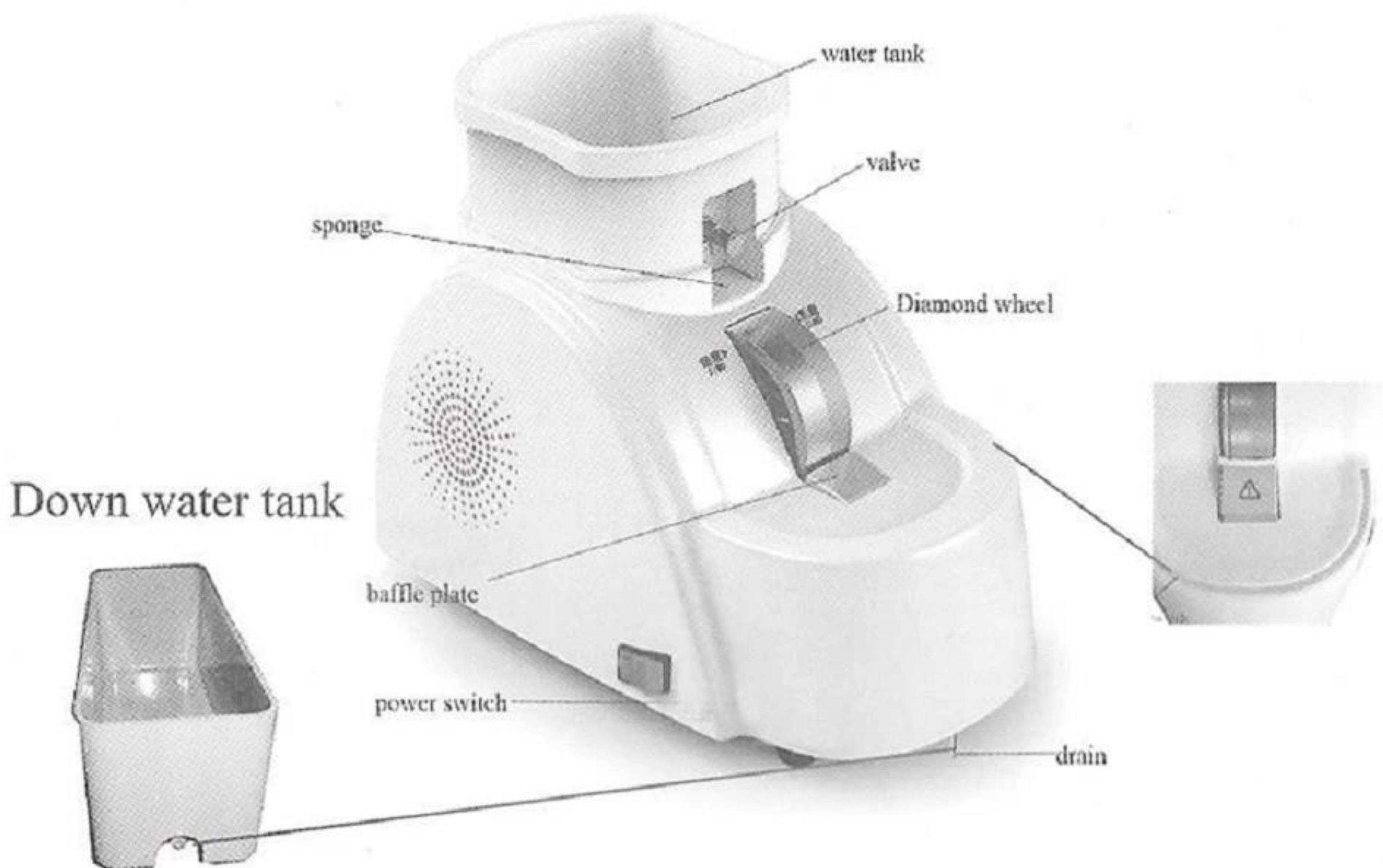

# Safety Precautions

## [Warning]

Don't operate the machine before reading thoroughly and grasping all the instructions and rules. Unknown the rules stated in the manual may cause accidents such as fire, shock personal injury etc.

In order to operate the machine and instruct others on the safe side all the time, please read this manual over and over.

Note: The manufacturer of the machine is out of responsible or obliged to any accident caused by a careless operator who pays no attention to the rules of precaution related below.

## [Precautions]

- Don't use the machine for any other purpose except for grinding the edge of lens.
- While the machine is working, don't touch the wheel to avoid getting injured.
- When the electric power can not be reached (voltage and frequency) don't use the machine. You have to be clear about the particular condition of the power supply.
- Don't place the machine in high temperature or expose into strong sunlight
- If any liquid or other materials get into the machine, turn the switch off immediately and pull the plug out of the wall socket. It cannot be operated until an expert who have solved the problem.
- Except for original parts, do not equip other parts with the machine
- Don't work the machine over-load and please remove the plug out after long time no using.

---

HE-7000B    HAND EDGER